


“HOW DOES OCT MAKE A DIFFERENCE?”

“OCT INFLUENCES MY TREATMENT DECISIONS MORE THAN I EXPECTED,”



ANGIOGRAPHY UNDERESTIMATES MORPHOLOGICAL LESION SEVERITY AFFECTING VESSEL PREP AND STENTING STRATEGIES AS DEMONSTRATED IN THE LIGHTLAB CLINICAL INITIATIVE.¹

Performing intravascular imaging pre-PCI allows to properly assess lesion morphology and lesion severity to guide vessel prep decision-making to ensure optimal expansion, which has been shown to improve outcomes.^{2,3}

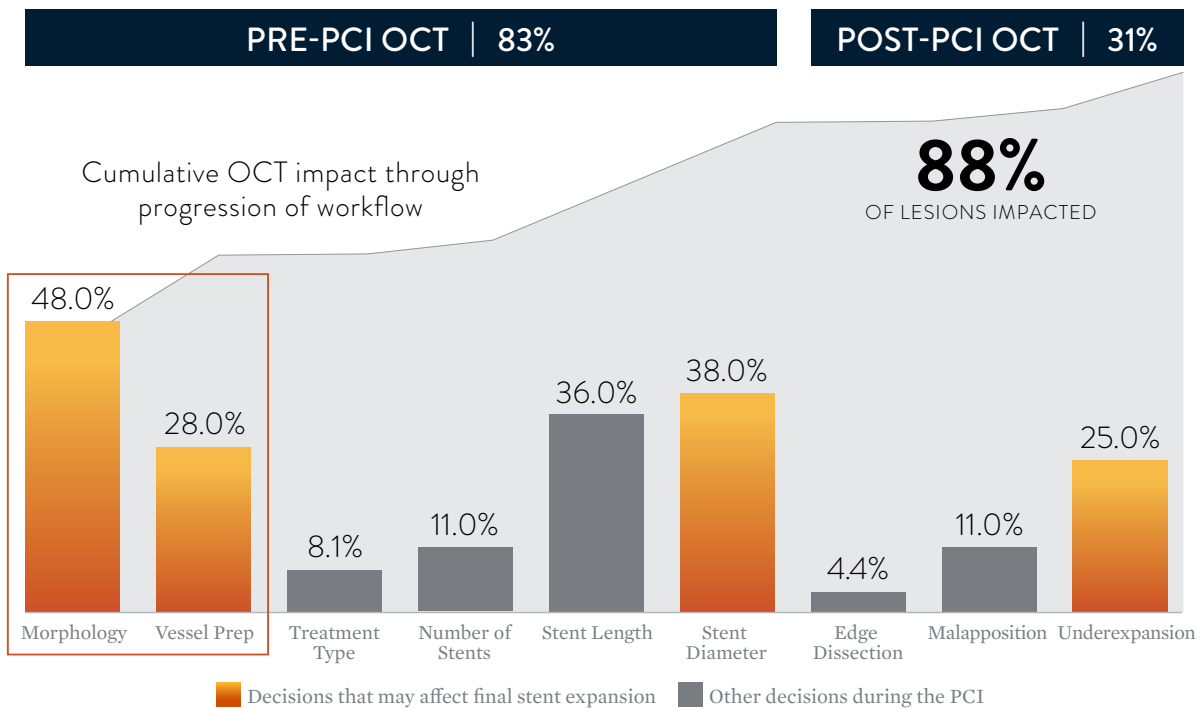


Physician assessment of lesion morphology with OCT changed in nearly **1/2** of lesions and changed vessel prep strategy in nearly **1/3** of lesions.¹

1. Bezerra, H. et al: Analysis of changes in decision-making process during OCT-guided PCI -Insights from the LightLab Initiative. EuroPCR2020 Presentation. 2. Zhang J, et al. Intravascular ultrasound versus angiography-guided drug-eluting stent implantation: the ULTIMATE trial. *J Am Coll Cardiol.* 2018;72(24):3126-3137 3. Hong M et al., IVUS-XPL 5 Year Outcomes, TCT 2019.

Information contained herein for DISTRIBUTION outside the U.S. only. Check the regulatory status of the device in areas where CE marking is not the regulation in force.



BETTER INSIGHTS. BETTER OUTCOMES.

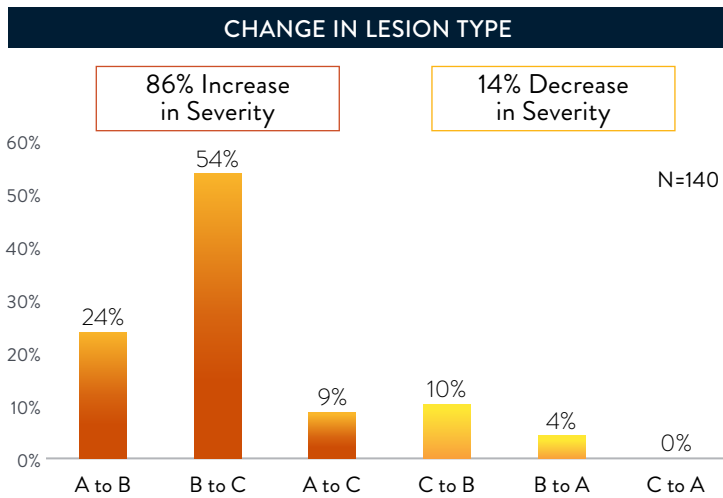

When a change in lesion type occurred (per ACC/AHA guidelines), **86%** of lesions had an increase in lesion severity.¹

When a change in vessel prep strategy occurred, calcification was the predominant morphology.¹

➔ Guiding PCI with the standardized OCT workflow, MLD MAX, changed over 80% of angio-based decisions on morphology, vessel prep and diameter, decisions which may influence final stent expansion. OCT reduces ambiguity, impacting decision-making so you can deliver optimal results.

Pre-PCI OCT | Strategize

Post-PCI OCT | Optimize

M	L	D	M	A	X
Morphology	Length	Diameter	Medial Dissection	Apposition	Xpansion
Search for High Calcium ²	Select Landing Zones Based on Healthy Tissue/EEL Visualization ³	Measure Vessel, Stent, Balloon Diameters ⁴	Address Significant Dissection ⁵	Address Gross Malapposition	Confirm Expansion ^{5,6}

1. Croce, K. et al. Optical Coherence Tomography Influences Procedure and Vessel Preparation Decisions During Percutaneous Coronary Intervention - Insights from the LightLab Initiative. *TCTConnect2020*. 2. Fujino, A. et al. A new optical coherence tomography-based calcium scoring system to predict stent under expansion. *EuroIntervention*, April 2018; 13(18):e2182-e2189. 3. Prati, F. et al. The CLI-OPCI II Study. *JACC: Cardiovascular Imaging*, 2015; Vol 8, No. 11:1297-305. 4. Kubo, T. et al. Application of Optical Coherence Tomography in Percutaneous Coronary Intervention. *Circulation Journal*, September 2012; Vol. 76, 2076-2083. 5. Ali, Z. et al. ILUMIEN III: Optimize PCI. *Lancet* 2016, 388:2618-2628. 6. Meneveau, N. et al. DOCTORS Study. *Circulation*, September 2016, 134:906-917; Zhang, J. et al. The ULTIMATE Trial. *Journal of the American College of Cardiology*, Dec 2018; Vol 72, No 24:3126-37; Russo, R. et al. The AVID Trial. *Circ Cardiovasc Intervent*, April 2009; 2:113-123.; De Jaegere, P. et al. MUSIC Study. *European Heart Journal*, February 1998;19,1214-1223.

CAUTION: This product is intended for use by or under the direction of a physician. Prior to use, reference the Instructions for Use, inside the product carton (when available) or at www.vascular.eifu.abbott or at medical.abbott/manuals for more detailed information on Indications, Contraindications, Warnings, Precautions and Adverse Events.

Illustrations are artist's representations only and should not be considered as engineering drawings or photographs. Photos on file at Abbott.

Information contained herein for DISTRIBUTION outside the U.S. only. Check the regulatory status of the device in areas where CE marking is not the regulation in force.

Abbott International BVBA
 Park Lane, Culliganlaan 2B, 1831 Diegem, Belgium, Tel: 32.2.714.14.11
www.cardiovascular.abbott
 ©2020 Abbott. All rights reserved. MAT-2009502 v1.0

